


How to access and view a CBCT scan sent via WeTransfer


Watch a how-to video at youtu.be/UitC3pwohrQ

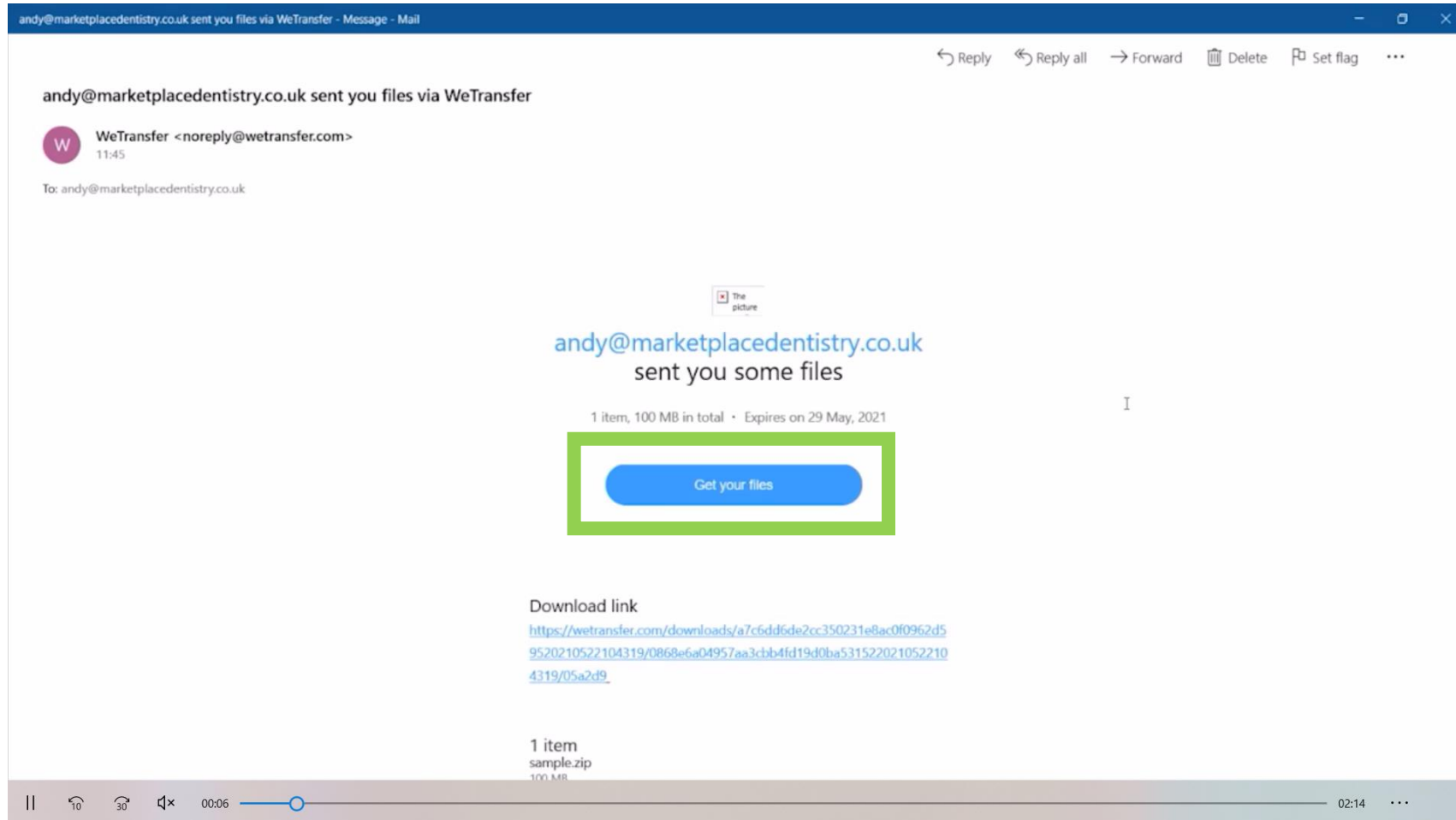


In the email you receive, click on the blue 'Get your files' button




In the WeTransfer browser page, click the 'Download' button. This downloads the CBCT scan as a .zip file

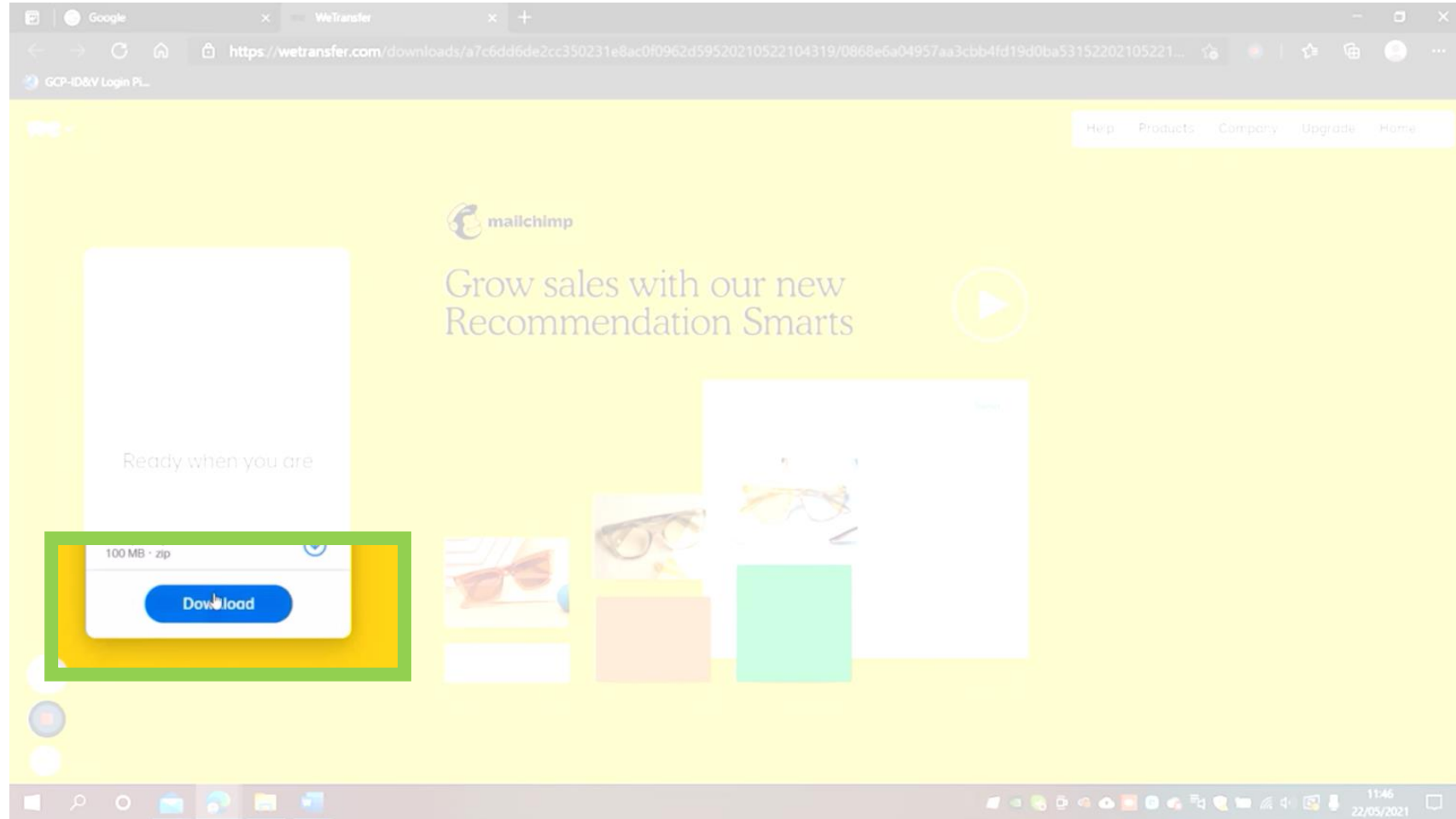


Open your Downloads folder to view the downloaded .zip file

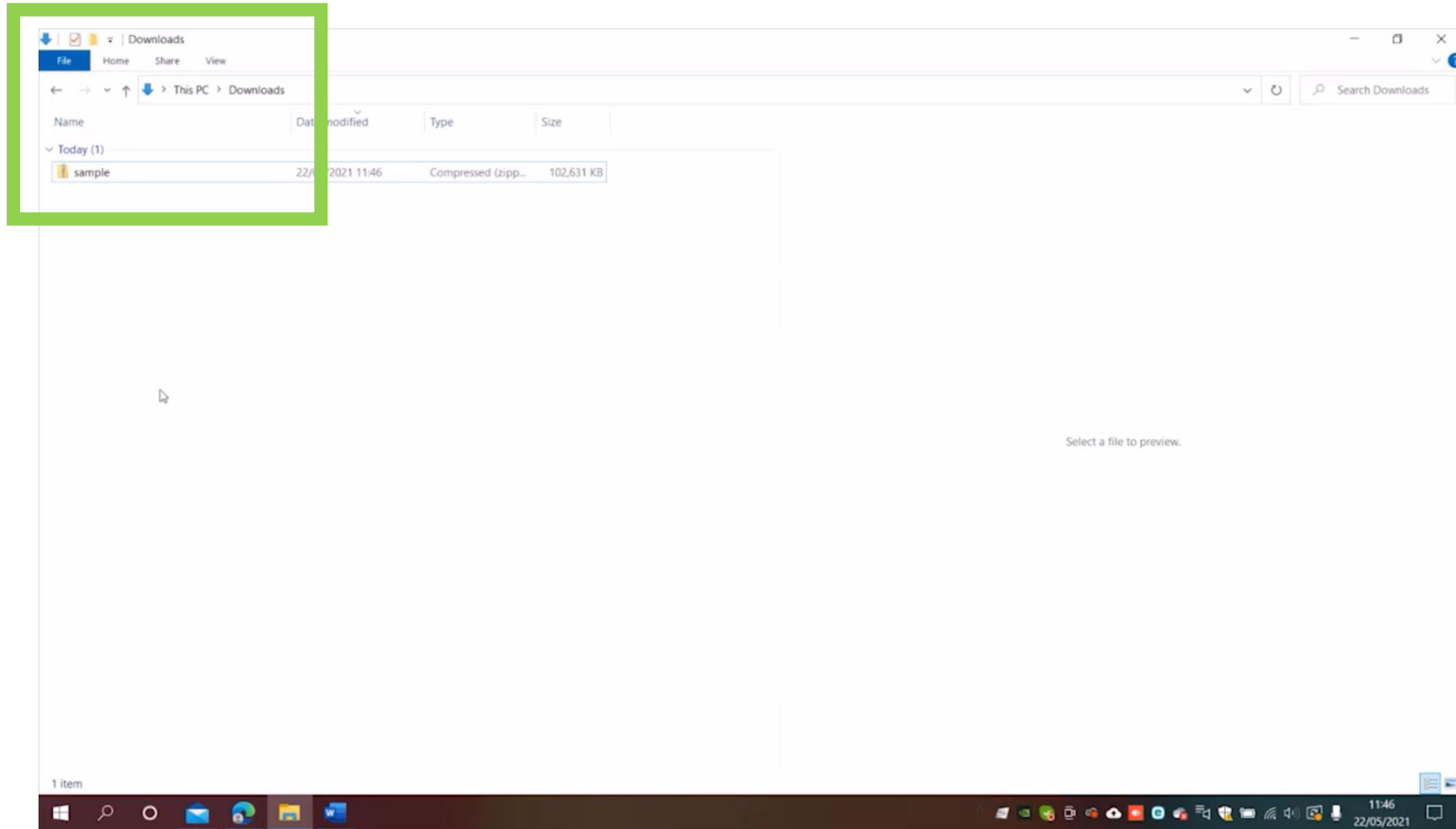


Right click on the .zip file and select 'Extract all'

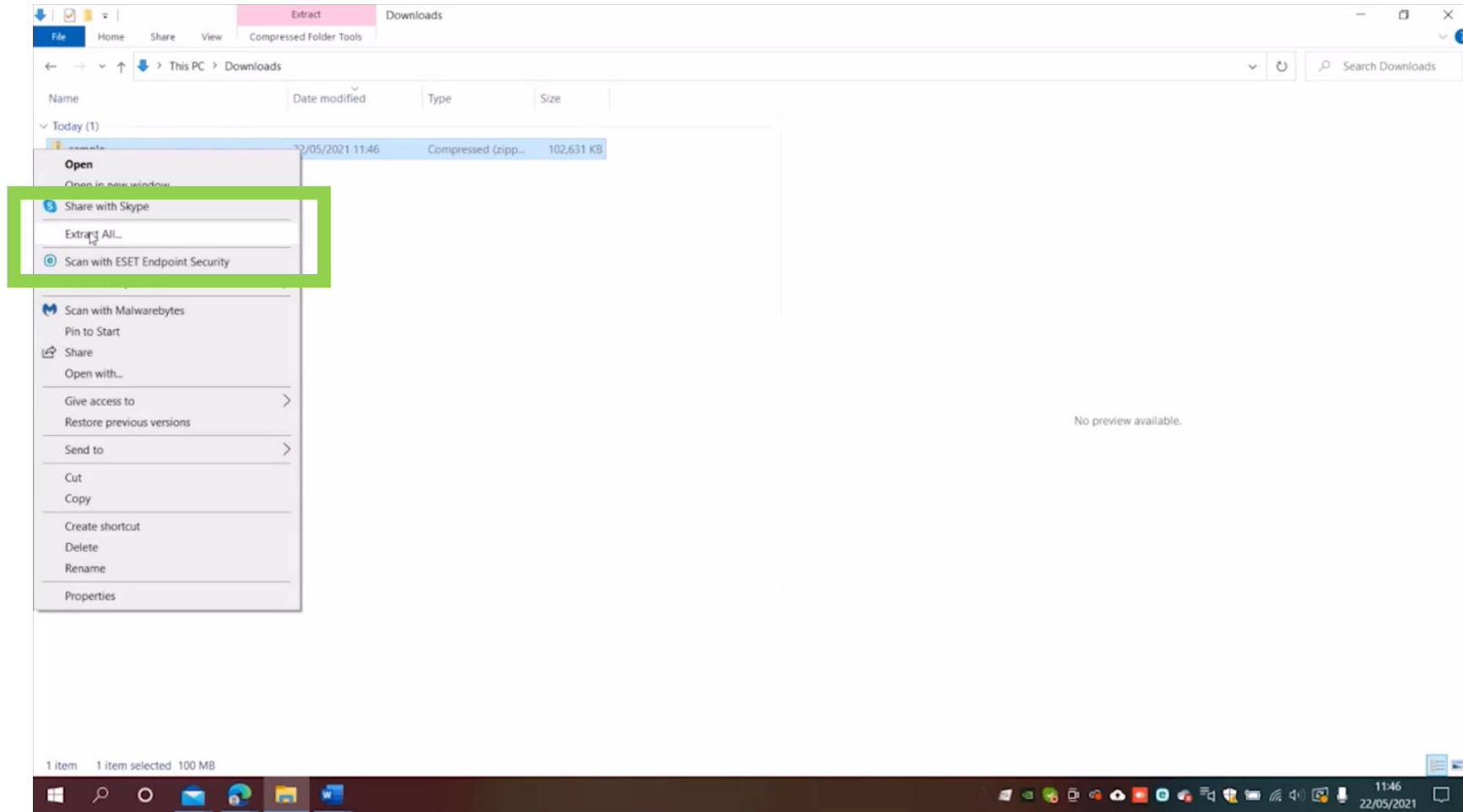


The password is your patient's DoB in the format DDMMYY

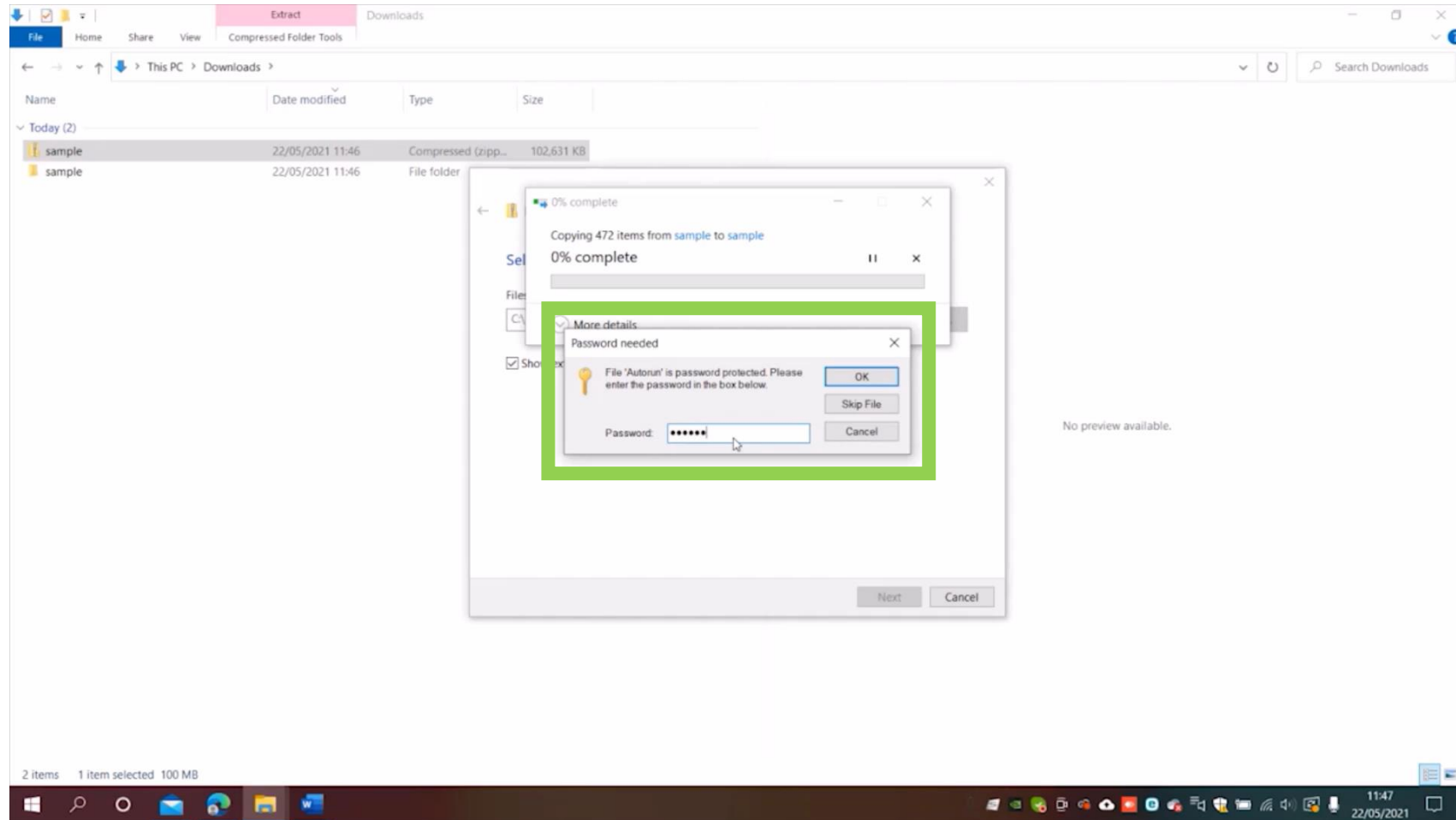


After extracting, open the folder and double-click 'ViewData'

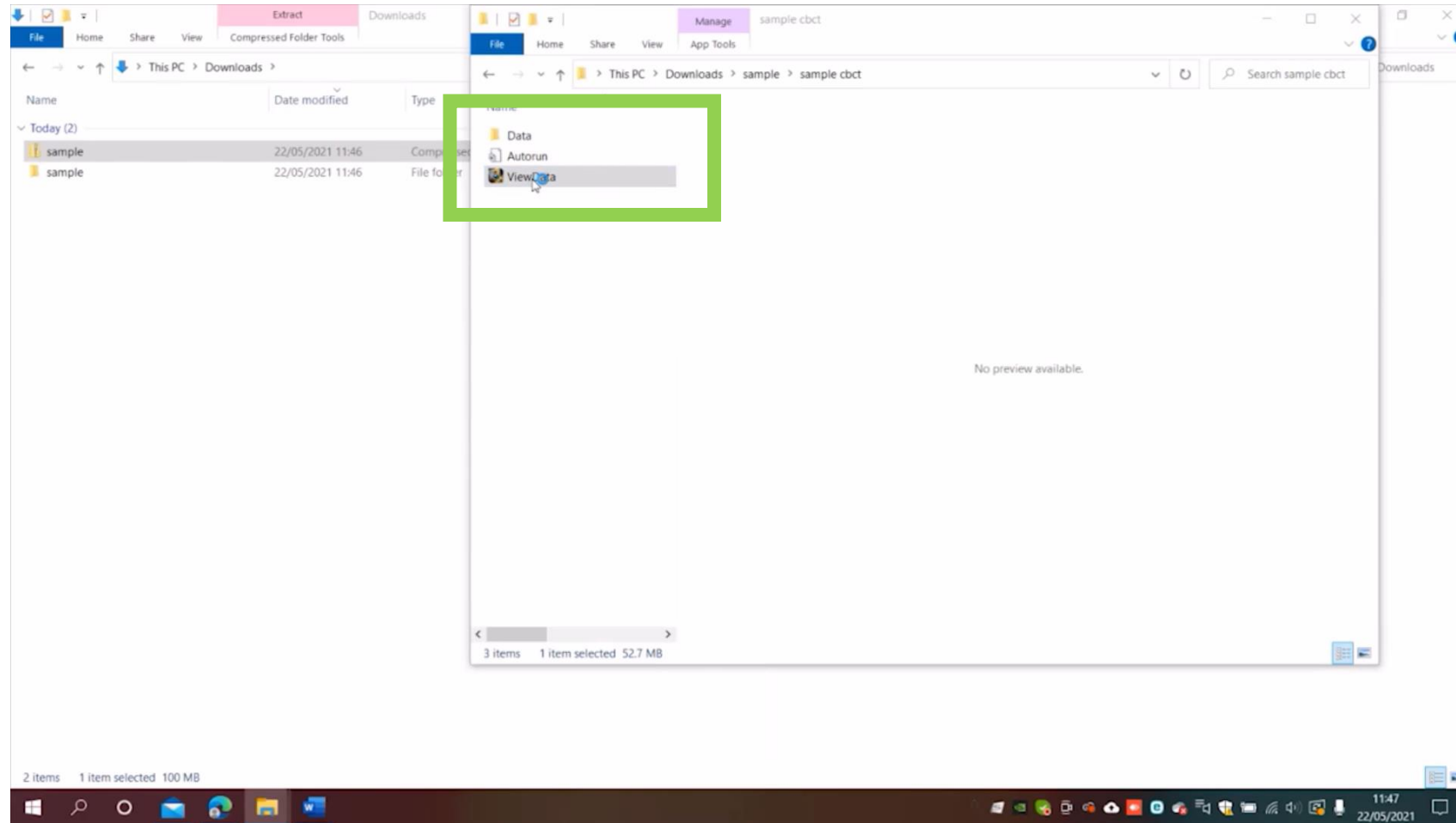


Click on the various 'Next' buttons to install the CBCT image viewer

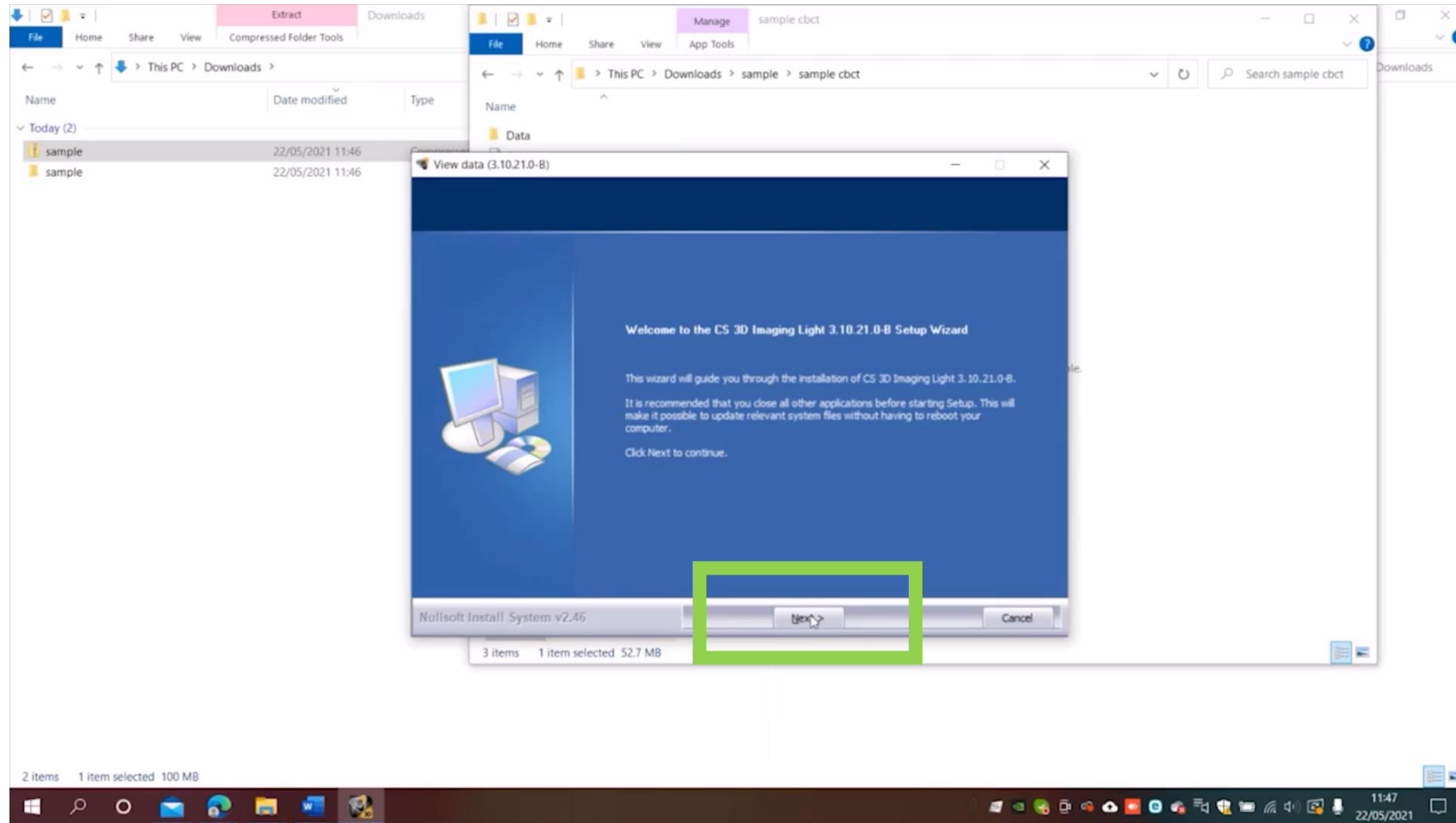


The Carestream viewing software should open automatically.
Click 'Accept'

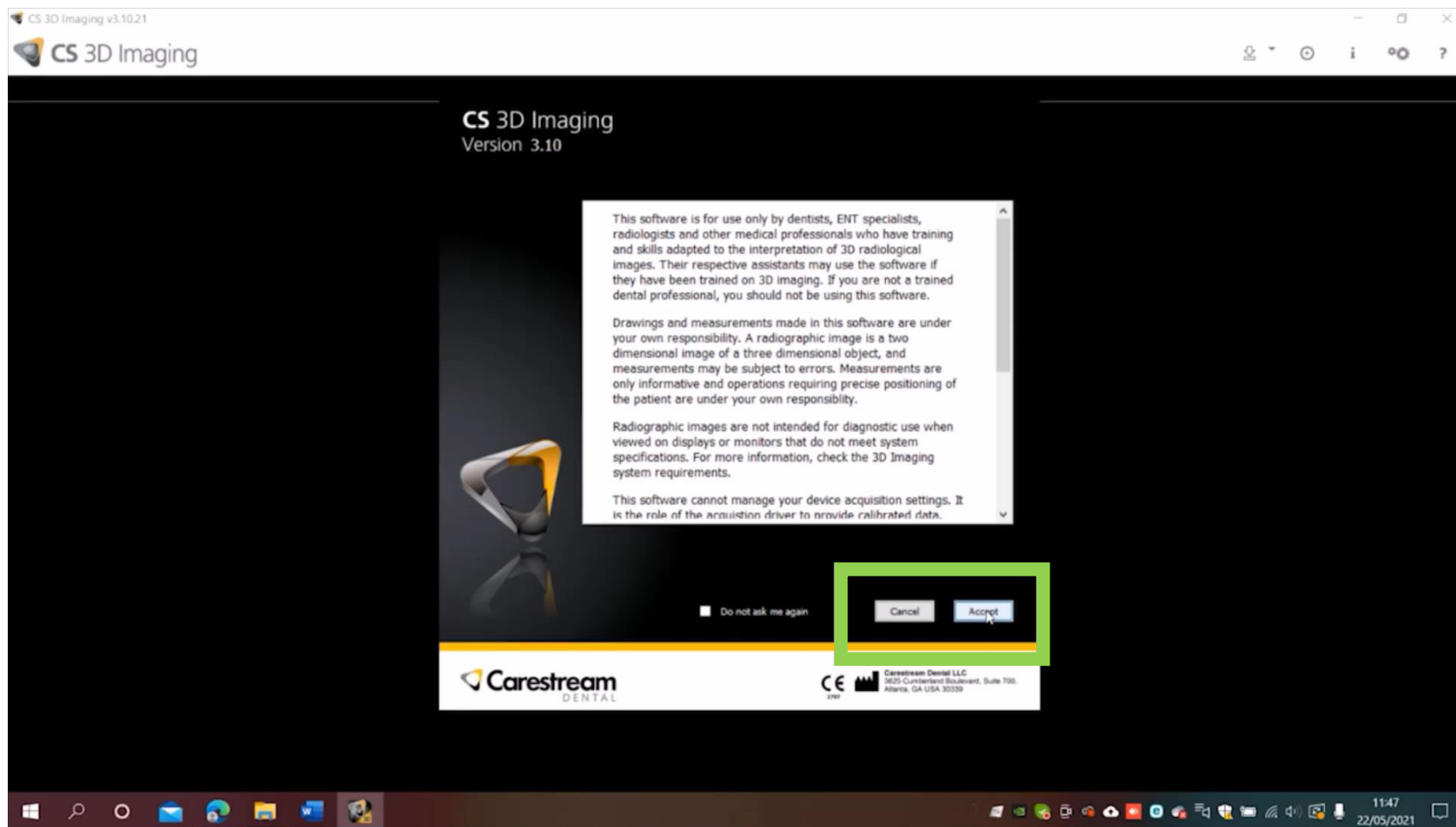

A window will open showing your patient's CBCT individual slices. Select the first slice and click 'Open'

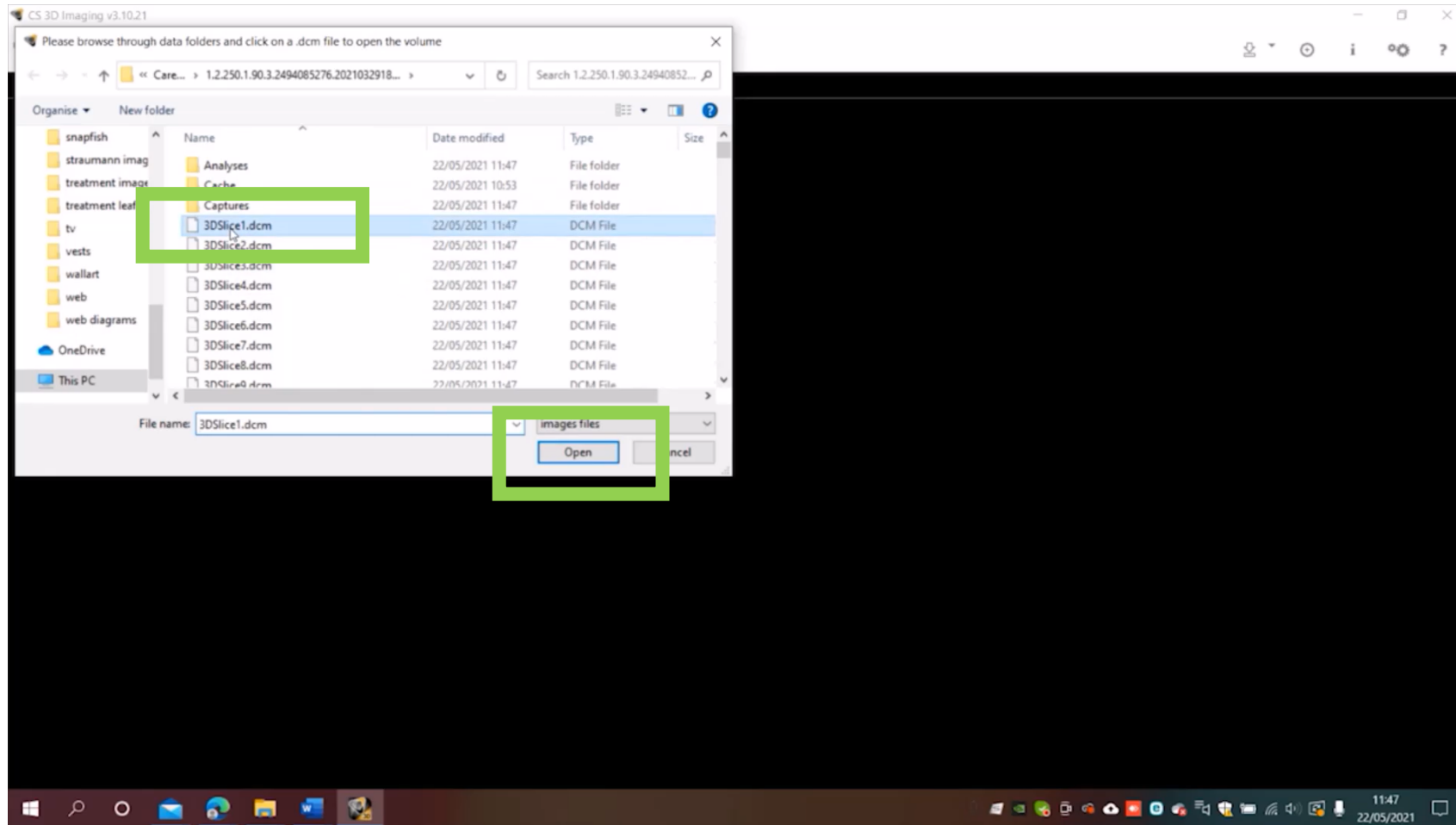

Your patient's CBCT scan will open showing the 3 Plane views

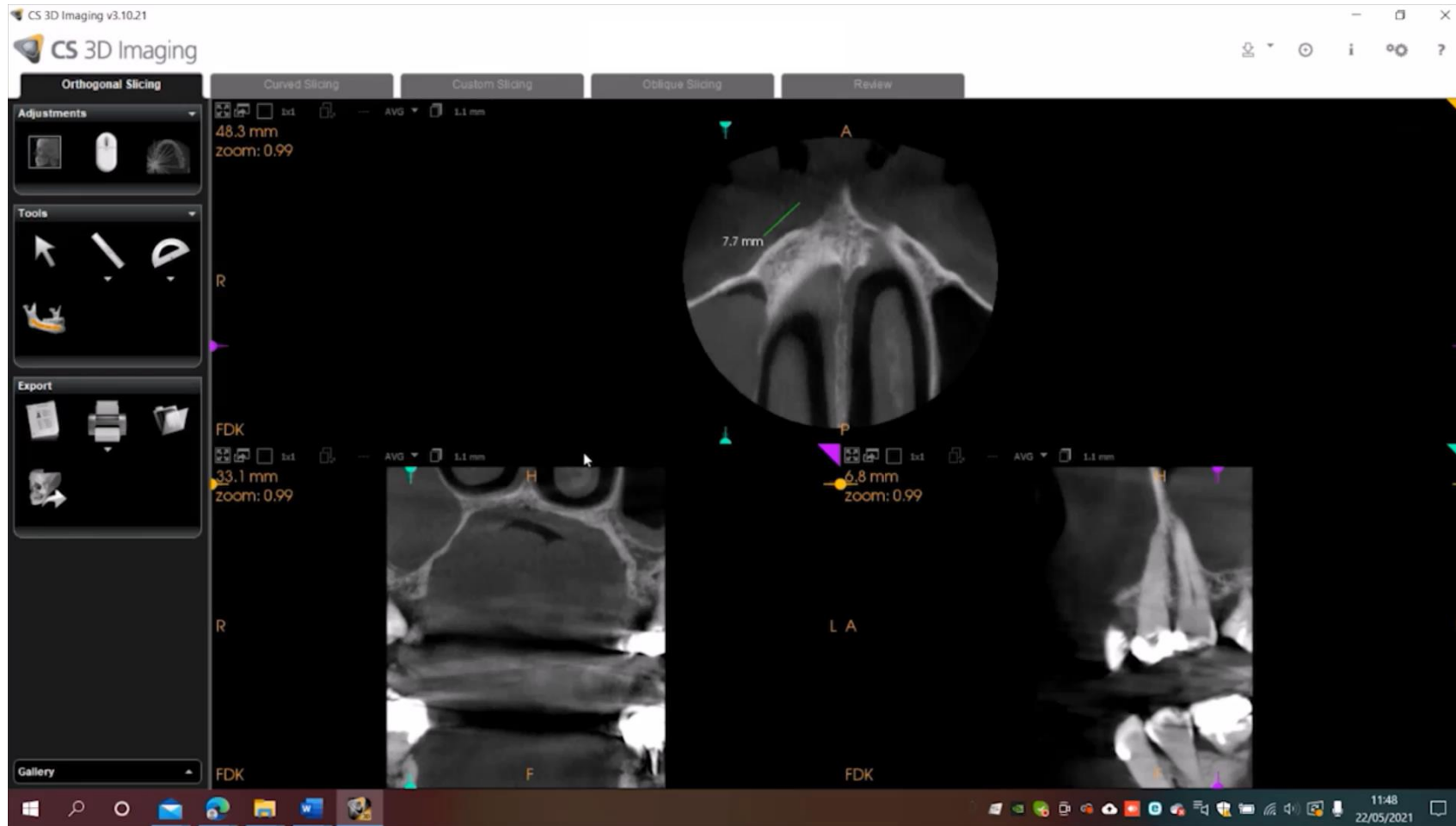

Click on the 'Curved Slicing' tab to open an OPT-type view
You can watch a software tutorial at youtu.be/-fDTpLjLORM

